Erkodur-al/-al 240

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EN: Instructions for use

Intended use: Thermoforming sheets are indicated for the fabrication of dental and orthodontic appliances for the treatment of all patient target groups.

Field of application: Erkodur-al is exclusively suitable for the fabrication of thermoformed intra-oral splints such as aligners (correction splints) and retainers.

Fabrication, see thermoforming technique brochure and videos, www.erkodent.com  

Contraindications: In case of already known allergic reaction to plastics a specific allergy to Erkodur-al (Copolyester) should be excluded.

Precautionary measures: Erkodur-al is only recommended for the above mentioned devices. Observe storage instructions.

Attention, sensitive to humidity: Close the aluminium bag immediately.

Attention: The use is limited to the production of dental and orthodontic appliances. Use is subject to the responsibility of the therapist. Use only on the recommendation or prescription of an authorised therapist. For single use only (thermoforming). Allergic reactions are unlikely, but cannot be excluded.

Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages. Serious incidents in connection with Erkodur-al must be reported by the user to the competent authority and to the manufacturer.

Erkodur-al (approx. values): density 1,19 g/cm³, modulus of elasticity 1462 MPa, elongation at break 179%, water absorption 0.5%, temperature resistance 85 °C

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